

Instruction Manual And Safety Instructions

Thank for your purchase of the DeBug Beach Wheelchair

We build the DeBug Beach Wheelchair with the highest quality materials, keeping in mind the comfort and positioning of the rider.

If any repairs should be required, please do not attempt them without proper technical training. Please contact your nearest distributor for any repairs, or adjustments needed

This Instruction Manual contains the following

Index		Page
1	Usage and Safety Information	2
2	Assembly/ Disassembly of DeBug	3-4
4	General Specifications and Warranty	5
5	General Maintenance Instructions	6

All information in this manual is subject to change without prior notice. Please contact your distributor for the latest version.

Thank-you
The De-Bug Mobility Products Team

Usage and Safety Information

Safety Warning

DeBug Beach Wheelchair should not be used during inclement weather. Please refrain from using during severe weather, dangerous surf and/or rough ocean currents, rip tides and any other poor conditions.

The DeBug is not intended to be used in the water. It is Not a floatation device! Avoid using during dangerous and or rough water conditions. These include, but are not limited to rip currents, waves, undertow etc. We recommended the chair be used only when a lifeguard is present. Safety comes first!

Users must always be accompanied by a responsible, able-bodied adult.

Using your DeBug Beach Wheelchair

Always maintain control of your DeBug while in use. When the DeBug is not in use, the safety brakes must be engaged to prevent the chair from moving unexpectedly.

The DeBug was built not to tip over, even when used over uneven surfaces. However, to ensure safety, and prevent injury, the user must always wear the seatbelt while using the DeBug This is imperative to the user's safety.

Do not use the components of the DeBug Beach wheelchair for any purpose other than what is intended. Do not have more than one person using the DeBug, at one time.

DeBug Beach Wheelchair Assembly Components



Always lock the wheels when transferring into or out of any wheelchair. I use a Transfer Sling for a single person transfer. For Safety, it is important that only professionally trained individuals assist in any "Wheelchair Transfer"

To Assemble the De-Bug Beach Wheelchair

***hand tighten all fasteners before any are secured**

1. Slide on the Leg Support Sling with the smooth side towards the skin
2. Slide on the foot plate, and tighten the Leg Length Adjustment knobs
3. Attach the Anti tip Wheels and tighten the four attaching 1/4-20x2” bolts
4. Install the Mid Axle, tighten the two attaching 1/4-20x2-1/2” bolts.
5. To install the Backrest, slide the backrest down on the chair frame and attach using the four Collar Clamps, align the clamps with the lines scribed on the Backrest Tubing.
6. The Rear Suspension Tension may be attached by inserting the Rear Suspension Tension bolt this bolt should be tight during use.

To Disassemble the De-Bug Beach Wheelchair.

1. Loosen the Leg Length Adjustment knobs and slide out the footplate.
2. Slide off the Leg Support Sling
3. To remove the Anti tip Wheels, loosen the four attaching 1/4-20 bolts and remove the wheelie bar.
4. To remove the Mid Axle, loosen the two attaching 1/4-20 bolts.
5. To remove the Backrest, loosen the four Collar Clamps and slide up on the Backrest.
6. The Rear Suspension Tension may be detached by removing the Rear Suspension Tension bolt. This bolt should be tight during use.

General Specification and Warranty Information

The De-Bug Beach Wheelchair frame is constructed using #316L Stainless Steel tubing, which is a higher grade than standard “#304 marine grade” stainless. Weight capacity 350 pounds.

The rear Caster wheels rotate 360 degrees. The wheel hubs are made from poly-nylon material to eliminate saltwater corrosion.

The De-Bug has a footrest for proper foot support.

Wheel Lock –Adjustable tension wheel brake can be actuated independently on each side,

Chair weight- ~40 pounds for a standard adult chair

Chair width is~33.5 inches to access a 36-inch doorway

30 DAY Money Back Guarantee

Ten-year warranty

Components & Frame #316L series Stainless Steel “the higher the number the higher quality of Stainless”

Two-year warranty

Seating-Material is Phiffertex Plus brand Nylon which is non-absorbent, durable and U.V. resistant. Seating is removable and machine washable.

Two-year warranty

Tires-3ply polyurethane, specifically designed to smoothly roll across sand, dirt, grass, and other uneven terrain

General Maintenance

It's so easy to maintain your DeBug Beach Wheelchair! Just follow these simple instructions, and you will be able to enjoy your chair year after year.

Rinse the chair off with clean water to remove any sand or debris after each use.

The sling material is removable and machine washable.

Check tire pressure before each use. These wheels are meant to be pliable and somewhat "Soft" to the touch. Recommended tire pressure is 2.5 to 4 PSI. These specifications are molded on each wheel.

If at any time you need to lubricate any part of the chair, DO NOT USE WD-40, as it will attract dirt and sand causing possible damage to the wheel bearing/bushings. We recommend P.T.F.E Dry Lube, on all wearing components. This dry film is a superior lubricant and repels debris and water, Marine and Performance parts stores likely stock this locally.

Have any questions, or need additional support? Please don't hesitate to contact us!

Phone 850-478-5765

Email kmdeming@aol.com

Finally, we love to hear, and see how much you are enjoying your DeBug Beach Wheelchair. Feel free to email us any pictures, or videos, or post them to our Facebook page.