



Instruction Manual And Safety Instructions

Thank for your purchase of the EZ Roller Beach Wheelchair.
We build the Ez Roller with the highest quality materials, keeping in mind the comfort and positioning of the rider.
If any repairs should be required, please do not attempt them without proper technical training. Please contact your nearest distributor for any repairs, or adjustments needed

This Instruction Manual contains the following

Index		Page
	1	Usage and Safety Information
	2	Assembly/Dissassmbly of Ez Roller
	3	Installation/Removal of Pontoons
	4	General Specifications and Warranty

All information in this manual is subject to change without prior notice. Please contact your distributor for the latest version.

Thank-you
The De-Bug Mobility Products Team

Usage and Safety Information

Safety Warning

The EZ Roller should not be used during inclement weather. Please refrain from using during severe weather, dangerous surf and/or rough ocean currents, rip tides and any other poor conditions.

To ensure their safety, all users should wear a lifejacket when using the EZ Roller while in the water. The EZ Roller is not intended to be for dangerous and or rough water conditions. These include, but are not limited to rip currents, waves, undertow etc. We recommended the chair be used only when a lifeguard is present. Safety comes first!

Users must always be accompanied by a responsible, able-bodied adult.

Using your EZ Roller


Always maintain control of your EZ Roller while in use. When the EZ Roller is not in use, the safety brakes must be engaged to prevent the chair from moving unexpectedly.

The EZ Roller was built not to tip over, even when used over uneven surfaces. However, to ensure safety, and prevent injury, the user must always wear the seatbelt while using the EZ Roller. This is imperative to the user's safety.

Do not use the components of the EZ Roller for any purpose other than what is intended. Do not have more than one person using the EZ Roller at one time. When using the detachable pontoons, do not sit on them, or use them to lift the chair


EZ Roller Assembly Components

1. 1/4"-20x2-1/4"SS welded Hex Bolt with Nylon Insert Nut, 4ea
2. 1/4"-20x1-1/4"SS Phillips Bolt with Cap Nut, 8ea
3. 1/4"-20x1-1/4"SS Socket Cap Bolt with Cap Nut, 4ea
4. 1/4"-20x1-1/4"SS Hex Bolt with Nylon Insert Nut, 2ea
5. 1/4" x1-1/4" SS Marine Quick Release Pin, 4ea
6. 5/16"-18x2-1/4" SS Hex Bolt with Nylon Insert Nut & Nylon Washers, 4ea
7. 3/8"-16x2-1/4" SS welded Hex Bolt with Nylon Insert Nut, 2ea
8. *Note Pontoon Mount installed under Chair Frame for 42 cm Main Wheel
9. ~Note Brake Clamp Orientation is mirrored when using 42cm vs. 49cm Main Wheel
- 10.# Setup with 42 cm Main Wheel

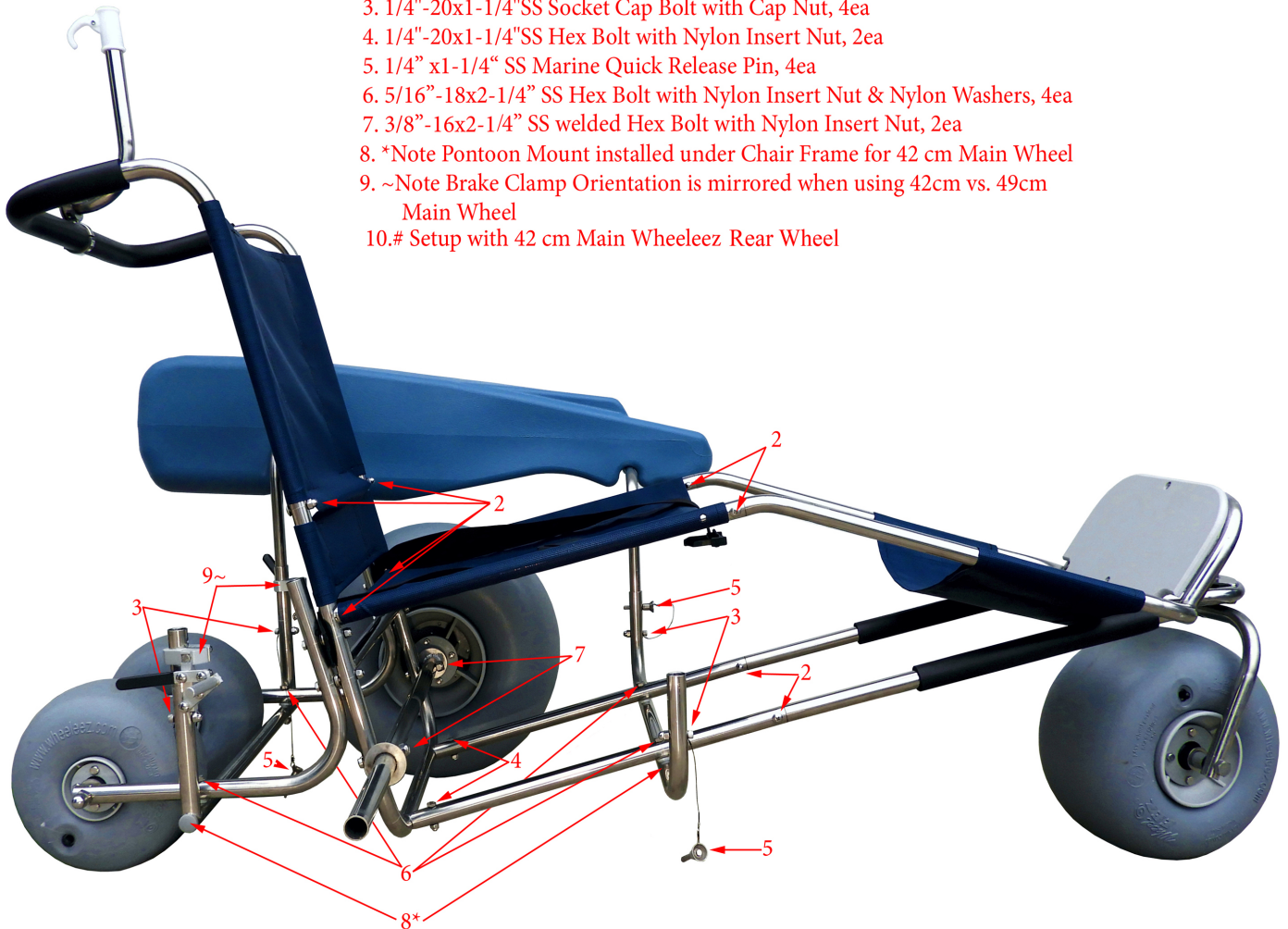

1. 1/4" 20x2-1/4"SS welded Hex Bolt

Pontoon Assembly Instructions

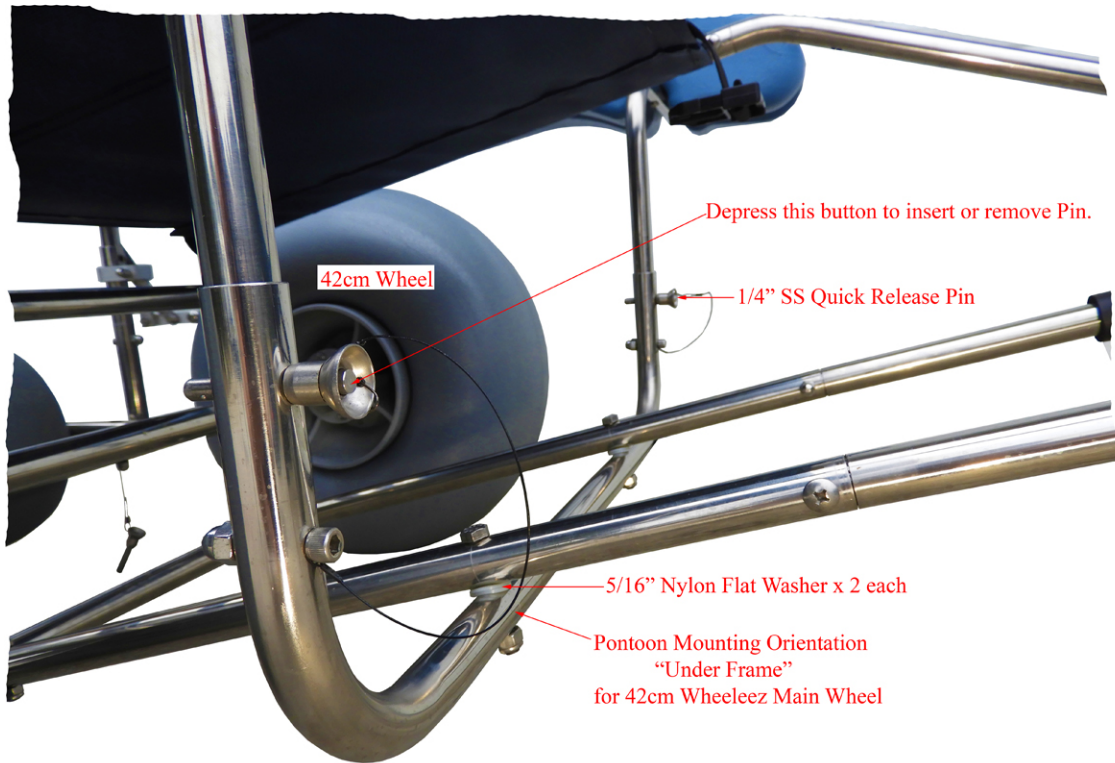

General Specification and Warranty Information

The EZ-Roller frame is constructed using #316L Stainless Steel tubing, which is a higher grade than standard “#304 marine grade” stainless.

Weight capacity 350 pounds.

The front Caster wheel rotates 360 degrees. The wheel hub is made from poly-nylon material to eliminate saltwater corrosion.

The EZ Roller has a footrest for proper foot support.

Wheel Lock –Adjustable tension wheel brake can be actuated independently on each side,

Chair weight-Forty pounds for a standard adult chair

Chair width is~33.5 inches to access a 36-inch doorway

30 DAY Money Back Guarantee

Ten-year warranty

Components & Frame #316L series Stainless Steel “the higher the number the higher quality of Stainless”

Two-year warranty

Seating-Material is Phiffertex Plus brand Nylon which is non-absorbent, durable and U.V. resistant. Seating is removable and machine washable.

Two-year warranty

Tires-3ply polyurethane, specifically designed to smoothly roll across sand, dirt, grass, and other uneven terrain

General Maintenance

It's so easy to maintain your EZ Roller beach wheelchair! Just follow these simple instructions, and you will be able to enjoy your chair year after year.

Rinse the chair off with clean water to remove any sand or debris after each use.

The sling material is removable and machine washable.

Check tire pressure before each use. These wheels are meant to be pliable and somewhat "Soft" to the touch. Recommended tire pressure is 2.5 to 4 PSI. These specifications are molded on each wheel.

If at anytime you need to lubricate any part of the chair, DO NOT USE WD-40, as it will attract dirt and sand causing possible damage to the wheel bearing/bushings. We recommend P.T.F.E Dry Lube, on all wearing components. This dry film is a superior lubricant and repels debris and water .

Marine and Performance parts stores likely stock this locally.