

Welcome to the GRIT Freedom Chair: Pro

Congratulations on your new GRIT: Pro! We hope the following pages will help you get your adjustable seat back installed and adjusted correctly.

The standard User Manual will be the best resource for most setup and general-use questions. For help installing the other components of your chair, see Chapter 2 of that User Manual. For Riding Tips, see Chapter 3. For Maintenance and Warranty Information, see Chapter 4.

See the following two pages for help with your GRIT: Pro seat back!



The GRIT customer support team is always happy to help. Give us a call at 617-356-8106 or email us at support@gogrit.us.

GRIT: Pro seat back installation

Have a question? Give us a call at 617-356-8106 or email us at support@gogrit.us.


1

Make sure the clamps on the vertical receiving tubes (on the seat pan) are loosened.



2

Align the canes of the seat back with the receivers in the seat pan.

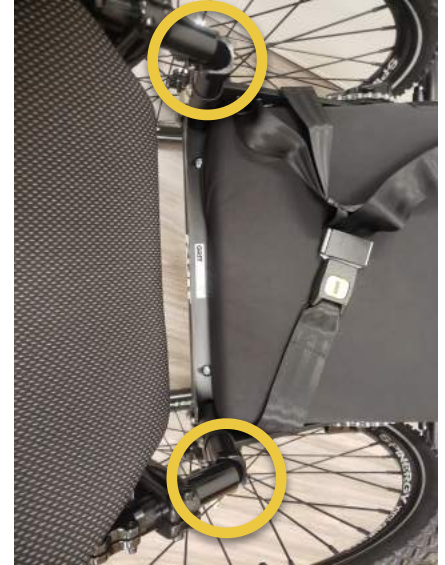

3

Fully insert the canes of the seat back into the receivers.



4

Secure the seat back by closing the seat back clamps.



GRIT: Pro seat back adjustments

Have a question? Give us a call at 617-356-8106 or email us at support@gogrit.us.


The GRIT: Pro has an adjustable seat back. Here's how to adjust it!

To adjust your GRIT: Pro seat back, use a 5mm Allen key to loosen the three bolts shown in the photo to the right. Then, loosen the three identical bolts on the other side of the seat back.




Adjustment option #1:

Once the bolts are loosened, you can slide the entire seat back forward or backward.

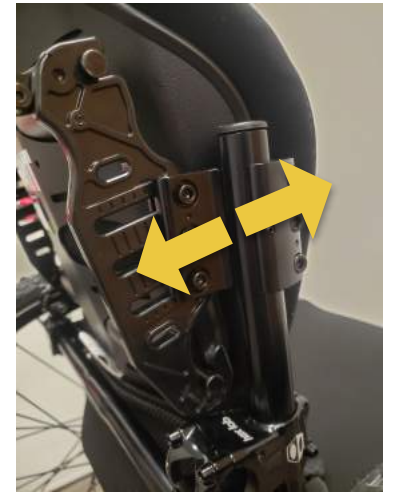

Adjustment option #2:

You can also shift the bottom of the seat back farther forward than the top (or vice versa).

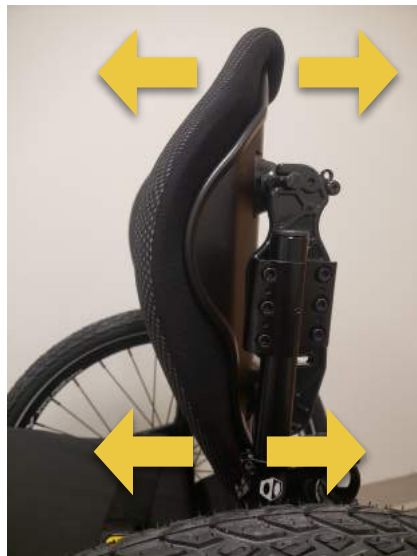

Adjustment option #3:

Finally, you can skew the right or left side of the seat back forward.



These bolts allow you to move the seat back up and down the canes. If loosening these bolts, be extra careful to keep the seat back centered on the canes!

Note:

