

Replacing the Lever Couplings

Have a question? Give us a call at 617-356-8106 or email us at support@gogrit.us.

Tools Required:

- 5mm hex key

Installation Instructions:

1. Remove rear wheels and gently place the Freedom Chair on its side. Lay a clean towel down to keep the moving parts of the chair clean.

2. On the side of the chair facing upward, locate the 3 screws keeping the lever coupling in place.

3. Use the 5mm hex key to fully remove the three highlighted screws. The coupling will then come loose. Place the three bolts safely aside.

4. Place the new coupling on the gear, aligned with the 3 screw holes.

NOTE: Use the L/R stickers on the new coupling to identify which side the coupling should go on.

5. Use the 5mm hex key to secure the screws and new coupling into place. Tighten the screws 1/4 turn at a time to ensure a level attachment.

6. Repeat the above for the other side of the chair.

NOTE: It can be useful to check your chain tension after swapping the Lever Couplings. Go to www.gogrit.us/freedom-chair-support#chaintensionvideo for instructions and tips.

