

# Parking Brake Installation

---

Have a question? Give us a call at 617-356-8106 or email [support@gogrit.us](mailto:support@gogrit.us).

Riders of our 20" seat width chair need to install their own parking brakes before riding their new Freedom Chair. It's a quick process and is easiest to perform before attaching the wheels.

## 1

The right and left brakes are labeled "R" and "L." Remove the screw, then slide them onto their corresponding sides (from the rider's perspective) of the chair. Be sure that when the brake lever is hanging down, the two brake bars are facing the front of the chair, as shown in the photo.



## 2

Carefully thread the screw into the chair from the bottom. Make sure it engages the threads, then use the provided Allen key to tighten it in place. Make sure it is tight and all of the way in. That's it!

