

Side Guard Installation Guide

Have a question? Call 617-356-8106 or email support@gogrit.us.

The Side Guards have two different setups: Removable and fixed. In the removable setup, magnets hold the Side Guards in place, making it easy to disassemble the chair for transit. In the fixed setup, screws hold the Side Guards in place, enabling more secure attachment. Both Side Guards are identical: There is no left or right.

For Removable Side Guards


1. Leave the screws in the bag. You won't need them for this setup, but do save them in case you switch to fixed Side Guards.
2. Slide the Side Guard into place between the seat and the cushion. Optionally, remove the seat cushion while putting on the Side Guard. Magnets will hold it in place.



Shown with seat cushion removed.

For Fixed Side Guards

1. Remove screws and washers from bag.
2. Slide the Side Guard into place between the seat and the cushion (remove cushion for easier access).
3. Insert the screws through the washers into (any) three holes in the seat pan, carefully threading them into the Side Guard. Position the screws so the bolt heads are on the outside of the seat pan.

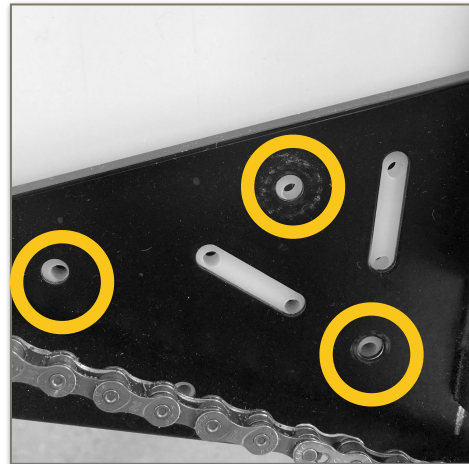

Highlighted: three potential holes to insert screws.



Finished look at installed Side Guard.