USER GUIDE 1-6

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COMODITÀ

Excellence in design and safety

Spazio *special*

EXTRA WIDE
ROLLING WALKER
WITH ORTHOPEDIC SEAT

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Comodità is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

The modern design and sturdy construction of the Comodità Spazio *special* Rolling Walker make it one of the safest walkers on the market. Built to support weights of up to 440 lbs (200 kgs), it is tested and approved under the most demanding standards for rolling walkers.

The unique design of the frame provides extra room for the feet to move, helping to avoid tripping and falling. It also allows the weight to be placed right above the wheels, and not on the side of the wheels like most models on the market. Our rolling walker is built with the best materials available to ensure the highest safety you can find.

Comodità Spazio *special* wheels are placed directly under the frame, allowing the full weight of the user to be placed directly on the wheels.

On other models the wheels are attached to the frame on the side, which is less supportive and less stable.

Please do not hesitate to contact us by e-mail at info@comoditusa.com or through website www.comoditusa.com if you have any doubts, questions or problems. We are here to help!

Other Features:

- Reinforced frame for maximum security
- Adjustable handle height
- Extra-large 8" wheels for safer indoor and outdoor use
- Rubber tires provide superior grip when brakes are applied, even on smooth indoor surfaces like ceramic tiles, wood and carpet.
- Hand brake locks for safety while seated
- Comfortable padded nylon seat
- Padded ergonomic backrest
- Convenient under seat storage bag

Product Dimensions and Specifications:



- Height (min /max)... 35" / 39"
(89 cm/99 cm)
- Length..... 28" (71 cm)
- Width..... 28" (71 cm)
- Seat Depth.... 14 ¼" (36 cm)
- Seat Width.....20" (51 cm)
- Seat Height....23 (58 cm)"
- Folded Width... 10" (25.5 cm)
- Wheel Diameter.... 8" (20 cm)
- Unit Weight..... 22 lb (10 kg)
- Weight Capacity.... 440 lb (200 kg)

Assembly Instructions:

Assembling your Comodità Spazio special Rolling Walker is extremely simple!

1. Carefully remove the walker from the shipping carton.
2. Unfold the walker by pushing down the folding brackets on both sides of the rolling walker until they are fully extended.
3. Insert handle bars into the frame of the walker, keeping the brake cables outside the frame.
4. Lower the backrest.
5. Adjust the height of the handles by standing upright behind the walker with your shoulders relaxed and arms hanging at your side. The handgrips should be at a height even with your wrists.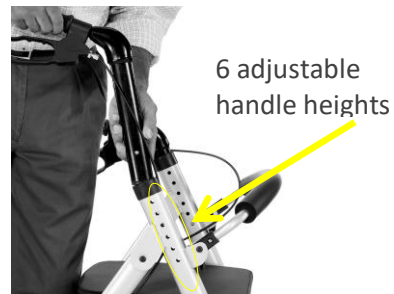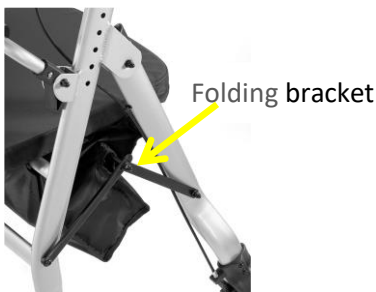

It's that simple. No tools, no loose parts.

Operating Instructions:

To Walk

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, pull the brake handles up.

To Lock and Unlock Brakes

- To lock the brakes, push down on the brake handles until the brakes click into locked position, as shown below. Do not push it any further than this point.
- To unlock the brakes and resume walking, pull up the brake handles until you hear a "click."

Brake in
unlocked
position

Push down
to lock brake

To Use as a Seat:


- Make sure your walker is on a flat surface, not on an incline.
- **Lock** the brakes before sitting.
- Make sure your feet rest flat on the ground.

Care & Maintenance:

Periodic Brake Adjustment

To adjust the sensitivity of the brake:

- Turn locking nut clockwise to reduce brake sensitivity.
- Turn locking nut counter-clockwise to increase sensitivity

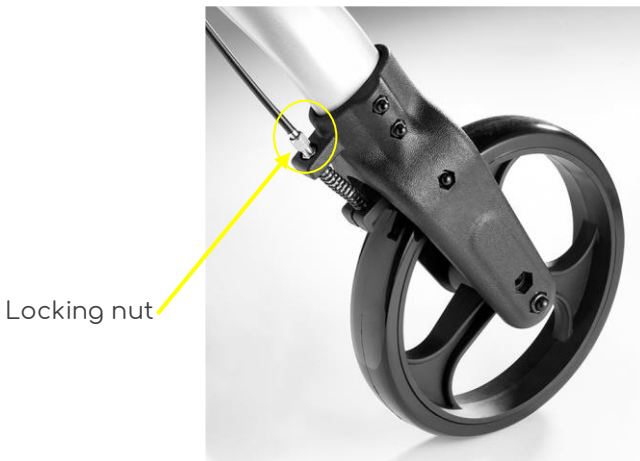Warning!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- *Do not use walker if brakes are not working properly.*

Safety Instructions:

Warning! Failure to follow safety instructions may result in serious injury.

- DO NOT use this product as a wheelchair.
- DO NOT transport someone while sitting on this walker.
- DO NOT exceed maximum weight capacity.
- BRAKES MUST be in locked position before sitting on the seat.
- DO NOT self-propel the walker while seated.
- DO NOT sit on the walker on an incline.
- INSPECT BRAKES before each use.
- ALL WHEELS MUST be in contact with the floor or ground at all times.
- BACKREST IS NOT intended to support your full weight. It is intended to provide back support while seated.
- DO NOT use in escalators, stairs, moving sidewalks, or uneven surfaces.
- BOTH FEET should rest flat on the ground when sitting.
- RECOMMENDED USER'S HEIGHT from 5'4" to 6'4".

Warranty:

Your Comodità Spazio *special* Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase for the original owner. This warranty does not cover device failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges. Please contact our customer service at info@comoditusa.com. We will provide a return authorization number. You will be required to include a copy of your original receipt.

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COMODITÀ LLC USA