


Pro Trail Handles Installation Guide

Have a question? Call 617-356-8106 or email support@gogrit.us.

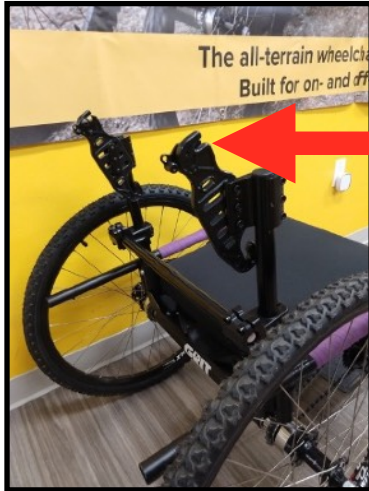

Seahorse clamp

1) Remove the seatback from the seahorse clamps.



Seahorse clamps removed

2) Use a 5mm Allen key to remove the seahorse clamps from the seatback canes.




Trail Handle stem clamp

3) Use a 4mm Allen key to open and then tighten the Trail Handle stem clamps onto the seatback canes.



5mm Allen key bolts

4) Use a 5mm Allen key to reattach the seahorse clamps to the seatback canes. Make sure the seahorse clamps are even on the canes.

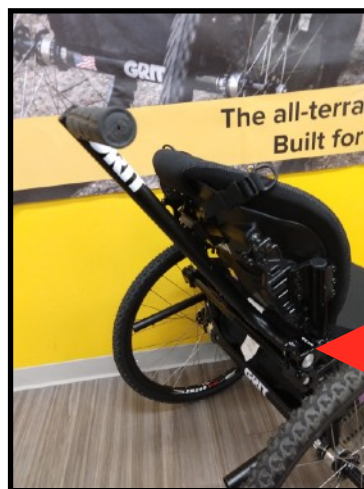

Insert Trail Handle here

5) Use a 4mm Allen key to open the stem clamp, insert Trail Handle, then tighten the clamp. Repeat for other Trail Handle.