USER GUIDE 1-10

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COMODITA

*Excellence in design and safety*

# Tipo

*stand up*

COMPACT UPRIGHT  
ROLLING WALKER

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COMODITÀ is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

The modern patented design and sturdy construction of the COMODITÀ Tipo Stand Up Rolling Walker (ref. COM 930) make it one of the safest and most stylish walkers in the market.

Built to support weights of up to 300 lbs., it is tested and approved under international testing standards for walkers and rollators.

The COMODITÀ Tipo Stand Up Rolling Walker is built with high-quality materials to ensure your safety.

In addition, the COMODITÀ Tipo Stand Up Rolling Walker comes with the exclusive COMODITÀ Pisa removable cup holder, cane holder and washable nylon shopping bag, as well as a convenient travel bag.

The COMODITÀ Tipo Stand Up Rolling Walker has two 10" aluminum front wheels, with pneumatic tires, and two 8" back wheels, also with pneumatic tires. That makes your COMODITÀ Stand Up perfect for indoor and outdoor use, on just about all kinds of surfaces.

**Please do not hesitate to contact us at [info@comoditausea.com](mailto:info@comoditausea.com) or through our website [www.comoditausea.com](http://www.comoditausea.com) if you have any problem with your COMODITÀ Stand Up. We are here to help!**

## **Main Features:**

- Extra-large 10" aluminum front wheels and 8" rear wheels with pneumatic tires, for safer indoor and outdoor use
- Wide, comfortable nylon seat
- Adjustable handle height
- Padded ergonomic backrest
- Convenient removable cup holder
- Cane holder included
- Convenient transport nylon bag
- Hand brake locks for safety while sitting
- Double-fold construction makes it very compact and easy to transport
- Four light reflectors at the front and two at the back for additional safety
- Nylon shopping bag with light reflecting straps for added security

## **Product Dimensions and Specifications**

- Unit weight: 23.5 lb (10.7 kg)
- Weight capacity: 300 lb (136 kg)
- Recommended user height: 5' to 6' (1,52 m to 1,82 m)
- Overall height: 40" / 46" (101.5 cm / 116.0 cm)
- Overall length: 28" (72 cm)
- Overall width: 25" (63.5cm)
- Folded measurements: 35"x 16"x 15"
- Front wheel diameter: 10" (25 cm)
- Back wheel diameter: 8" (20 cm)
- Seat height from floor: 21" (53 cm)
- Seat depth: 11" (28.5 cm)
- Seat width: 17½" (44 cm)
- Size of nylon bag: 17" W, 8" D, 10" H (43cm W, 20cm D, 25cm H)
- Recommended tire pressure: 20 PSI

## Assembly Instructions

Assembling your COMODITÀ Tipo Stand Up Rolling Walker is extremely simple! You can also watch the assembly video at [www.comoditauusa.com](http://www.comoditauusa.com) for more instructions.

1. Carefully remove the walker from the shipping carton.
2. Place the rollator on a flat surface

3. Raise the handles to the upright position.

4. Lock the frame on both sides by bringing the folding lock lever (pic 1 – make sure this lever is on this position) up 180 degrees, so it points up (pic 2). Turn the folding lever 90 degrees towards the back wheel, so it slides into the folding lock lever plastic support (pic 3). **Note:** You can only slide the folding lock lever into the plastic support if it is in the correct position, as seen in the pictures below.

PIC 1

PIC2

PIC 3

5. Insert both handles into the end of the backrest. Make sure the position of the backrest is correct and tighten the knobs.

6. Insert both handles into the frame of the walker and tighten the knob (see handle height adjustment below).

7. Release the plastic locking device that holds the walker folded.

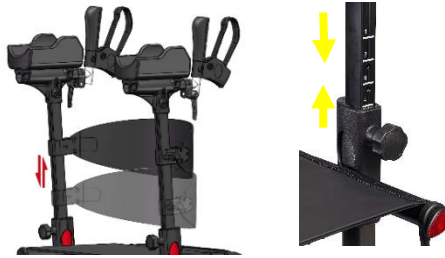
8. Open the walker and lock it in the opened position by pushing down the sides of the seat onto the main frame until you hear a click.

9. Adjusting the height of the handle:

- Untighten the plastic knob on the frame.
- Adjust the handle at the desired height.
- Tight the plastic knob on the frame to lock the handle at the desired height.

10. Adjusting the position of the handle:

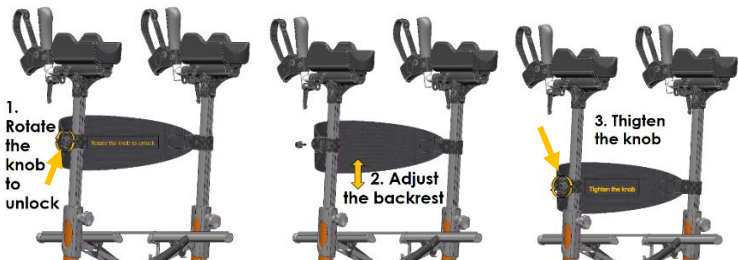
- You can adjust the brake handles vertical position to your most comfortable position by unlocking the locking knob.

- You can also adjust the distance of the brake handle from the arm support.

11. Adjusting the height of the backrest:

12. You will find a nylon strap on each side of the shopping bag that becomes a loop (fig 1). Locate the plastic supports on each side of the seat (Fig 2). Move the nylon strap loop around the plastic support (Fig3).

13. Secure the brake cables together in front of the walker. You will find a small Velcro strap that will allow you to tie the brake cables together.

14. Insert the cup holder into the plastic support located on the backrest.

15. Insert the cane holder into the plastic support

**IT IS THAT SIMPLE! NO TOOLS, NO LOOSE PARTS!**

## **Operating Instructions**

### **TO WALK:**

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, slowly squeeze the brake handles towards you.
- To lock the brakes, push forward the brake handles until the brakes “click” into locked position. Do not push it any further than this point.
- To unlock the brakes and resume walking, pull back the brake handles until you hear a “click.”

### **TO USE THE SEAT OF YOUR WALKER:**

- Make sure your walker is in a flat surface, not on an incline.
- LOCK the brakes before sitting.
- Make sure your feet rest flat on the ground.

## Care and Maintenance

### PERIODIC BRAKE ADJUSTMENT

To adjust the sensitivity of the brake:

- To tighten the brake (reduce the gap between the brake pad and wheel) rotate the adjusting nut clockwise.
- To loosen the brake (increase the gap between the brake and the wheel) rotate the adjusting nut counterclockwise.

### WARNING!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- **DO NOT USE WALKER IF BRAKES ARE NOT WORKING PROPERLY.**

## Safety Instructions

***WARNING!!! Failure to follow safety instructions may result in serious injuries.***

- **BE CAREFUL NOT TO TIP FORWARD YOUR COMODITÀ TIPO STAND UP WALKER.**  
COMODITÀ is very concerned with the safety of our products. That is why we only offer our Stand Up model with ultra large wheels with pneumatic tires. The 12" front wheels with **pneumatic** tires will allow you to enjoy your walker in many different terrains with less risk of tipping over.
- **DO NOT** walk too fast!
- **DO NOT** push your COMODITÀ Tipo Stand Up when you find an obstacle in the terrain! Go around the obstacle. **Trying to overcome an obstacle forcefully increases the risk of the walker tipping over.**
- **DO NOT** transport someone while sitting on this walker.
- **DO NOT** exceed maximum weight capacity.
- Brakes **MUST** be on **LOCKED** position before sitting on the seat.
- **DO NOT** self-propel the walker while seated.
- **DO NOT** sit on the walker on an incline.
- **INSPECT BRAKES** before each use.
- **DO NOT** use the walker if the brakes are not working properly.
- **ALL WHEELS MUST** be in contact with the floor or ground at all times.
- **BACKREST IS NOT** intended to support your full weight. It is intended to provide back support while you are seated.
- **DO NOT** use in escalators, stairs, moving sidewalks or uneven surfaces.
- **BOTH FEET** should rest flat on the ground when sitting.
- **ALWAYS** check if the seat is locked down and well connected to the frame when the rollator is open.

## Warranty

Your COMODITÀ Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase for the original owner. This warranty does not cover product failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our discretion, repair or replace the product. This warranty does not include any labor or shipping charges. Please contact our customer service at [info@comoditusa.com](mailto:info@comoditusa.com). We will provide a return authorization number. You will be required to include a copy of your original bill of sale.

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COMODITÀ LLC USA