

Utility Clamp with Foot Platforms Installation Guide

Have a question? Call 617-356-8106 or email support@gogrit.us.

1

Loosen and remove the four bolts on the Utility Clamp.



2

Lay the top half of the clamp (the half with the pedals/platforms) onto the boom of the Freedom Chair. Longer legs? Place the clamp farther from the seat.



Clamp positioned for shorter legs



3

Reinsert the four screws to join the two halves of the clamp. Tighten the bolts with a 5mm Allen key.





Note:

Tighten the bolts incrementally, in a criss-cross pattern. Uneven gap between the two halves of your Utility Clamp? Back the bolts out and retighten.

4

If using straps, loosen them and place one foot on its pedal/platform. Secure straps in this order: Wide velcro strap, two narrow straps, then top strap.

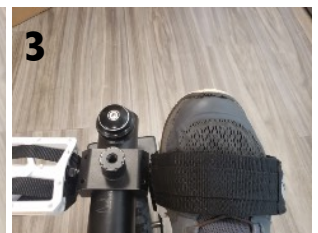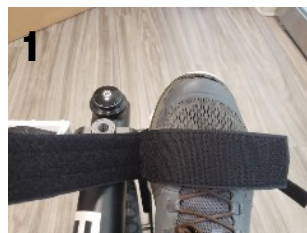

5

If applicable, repeat step 4 for your other foot. Remember: You can reposition the entire clamp along the boom, or adjust the velcro straps to place more or less of your feet on the pedals/platforms.

