

Utility Clamp with Leg Elevation Installation Guide

Have a question? Call 617-356-8106 or email support@gogrit.us.

1

Move the Utility Clamp to your desired position on the boom of the Freedom Chair. Tighten the four bolts of the Utility Clamp using a 5mm Allen Key.



2

Slide the stem clamp of the Leg Elevation accessory onto the vertical stem of the Utility Clamp. Tighten the two stem clamp bolts using a 5mm Allen Key.



3

If you'd like to adjust the position of the Leg Elevation bar, loosen the four Utility Clamp bolts, reposition the clamp, and then retighten the bolts.



Not using the standard foot rest?

Feel free to remove it altogether! This will only give your Freedom Chair more clearance over roots, rocks, curbs, etc. Otherwise, leave it right where it is and use it and the Leg Elevation interchangeably.