


Utility Clamp & Rope Mount Installation Guide

Have a question? Call 617-356-8106 or email support@gogrit.us.

Knob facing upward

1

Place the two halves of the Utility Clamp at the desired location on the Freedom Chair boom. Note that the rope mount should be on the top of the boom. Also make sure the “up” arrow is facing toward the rider (refer to 2nd image).

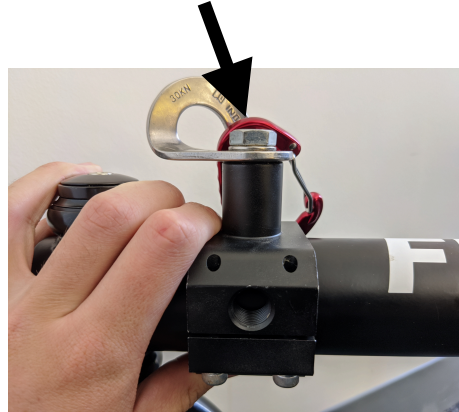

2

Secure the Utility Clamp using the four included bolts and a 5mm Allen key (you can hand-tighten them first). Tighten each bolt a bit at a time to make sure they are evenly tightened.



3

Secure your preferred pull-rope through the silver bracket. To adjust the placement of the mount, loosen the four screws and slide the clamp up or down the boom as desired.

